CE-KY-01-0202

SMS-01A-01

Urinary Catheters for Single Use (Foley)

[Product characteristic]

Direction for use

“KANGYUAN ” Urinary Catheters for Single Use (Foley) is made of imported silicon rubber by advanced technology. The product has smooth surface, slight stimulation, large apocensis volume, reliable balloon, convenient to use safely, multiple of types and specification for choice.

[Applicability]

The product can be clinically used to urinate and douche urinary bladder by inserting into urinary bladder though urethra.

[Specification]

REF	Tip	Size	Balloon (ml)	Length (mm)	Material	Lumen
ED-101003-SL	Nelaton, central hole	10Fr	Integrated	3	310	Silicone 2-way
ED-101205-SL	Nelaton, central hole	12Fr	Integrated	5	400	Silicone 2-way
ED-101405-SL	Nelaton, central hole	14Fr	Integrated	5	400	Silicone 2-way
ED-101610-SL	Nelaton, central hole	16Fr	Integrated	10	400	Silicone 2-way
ED-101810-SL	Nelaton, central hole	18Fr	Integrated	10	400	Silicone 2-way
ED-102010-SL	Nelaton, central hole	20Fr	Integrated	10	400	Silicone 2-way
ED-102210-SL	Nelaton, central hole	22Fr	Integrated	10	400	Silicone 2-way
ED-102410-SL	Nelaton, central hole	24Fr	Integrated	10	400	Silicone 2-way
ED-100803	Nelaton	8Fr	Integrated	3	310	Silicone 2-way
ED-101003	Nelaton	10Fr	Integrated	3	310	Silicone 2-way
ED-101205	Nelaton	12Fr	Integrated	5	400	Silicone 2-way
ED-101405	Nelaton	14Fr	Integrated	5	400	Silicone 2-way
ED-101610	Nelaton	16Fr	Integrated	10	400	Silicone 2-way
ED-101810	Nelaton	18Fr	Integrated	10	400	Silicone 2-way
ED-102010	Nelaton	20Fr	Integrated	10	400	Silicone 2-way
ED-102210	Nelaton	22Fr	Integrated	10	400	Silicone 2-way
ED-102410	Nelaton	24Fr	Integrated	10	400	Silicone 2-way
ED-440603	Nelaton	6Fr	Normal	1,5-3	310	Silicone 2-way
ED-440803	Nelaton	8Fr	Normal	3	310	Silicone 2-way
ED-441005	Nelaton	10Fr	Normal	5	310	Silicone 2-way
ED-441210	Nelaton	12Fr	Normal	10	400	Silicone 2-way
ED-441410	Nelaton	14Fr	Normal	10	400	Silicone 2-way
ED-441610	Nelaton	16Fr	Normal	10	400	Silicone 2-way
ED-441810	Nelaton	18Fr	Normal	10	400	Silicone 2-way
ED-442010	Nelaton	20Fr	Normal	10	400	Silicone 2-way
ED-442210	Nelaton	22Fr	Normal	10	400	Silicone 2-way
ED-442410	Nelaton	24Fr	Normal	10	400	Silicone 2-way
ED-401205	Tiemann	12Fr	Integrated	5	400	Silicone 2-way
ED-401405	Tiemann	14Fr	Integrated	5	400	Silicone 2-way
ED-401610	Tiemann	16Fr	Integrated	10	400	Silicone 2-way
ED-401810	Tiemann	18Fr	Integrated	10	400	Silicone 2-way
ED-402010	Tiemann	20Fr	Integrated	10	400	Silicone 2-way
ED-402210	Tiemann	22Fr	Integrated	10	400	Silicone 2-way
ED-402410	Tiemann	24Fr	Integrated	10	400	Silicone 2-way

[Direction for use]

- Lubrication: Generously lubricate tip and shaft of the catheter before insert.
- Insert: Carefully insert catheter tip into the bladder (normally indicated by urine flow), and then a further 3cm to ensure balloon is also inside it.
- Inflating water: **Using a syringe without needle, inflate balloon with sterile distilled water or 5% - 10% glycerin aqueous solution is supplied.** Recommended volume to use is marked on funnel of catheter.
- Extraction: For deflation , cut off the inflation funnel above the valve, or using a syringe without needle push into valve to facilitate drainage.

UT_IPU-TUL-EN_Rev2.0 (11-23)

CE-KY-01-0202

SMS-01A-01

5. Dwell catheter: the dwelling time is as the requirement of clinic and nurse.


[Contraindication]

The unsuitable condition considered by doctor.

[Precaution]

- Do not use ointments or lubricants having a petroleum base.
- Different specification of urethral catheter should be chose as different ages prior to use.
- This product had been sterilized by ethylene oxide gas, and discard after single use.
- If packing is damaged, do not use.
- The size and balloon capacity is marked on outer unit pack and funnel of catheter.
- The guide wire for the auxiliary intubation in the drainage channel of the catheter is pre placed in the children.
- In use, such as the discovery of urinary catheter, urinary extravasation, inadequate drainage, catheter replacement should be applicable specifications timely.
- This product should be operated by medical staff.
- Indwelling time recommend no more than 28 days.

[Symbol]

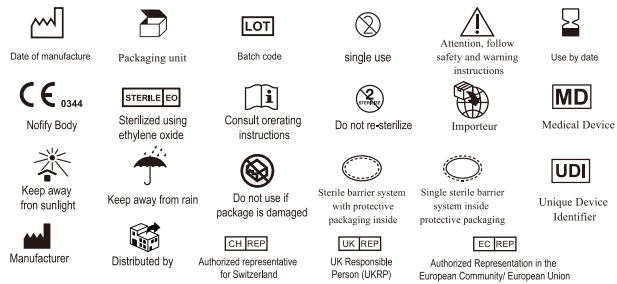[Warning]

The injection of sterile water shall not exceed the nominal capacity on the catheter (ml).

[Storage]

Store in a cool, dark and dry place, the temperature should not higher than 40 °C , without corrosive gas and good ventilation.

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www.urovision-urotech.de

[Registered person]

Manufacturers: HAIYAN KANGYUAN MEDICAL INSTRUMENT CO.,LTD.

Address: Songpodong Road, Shendang Town 314311, Haiyan, Zhejiang China

Tel: 0086-0573-86725881 0086-0573-86725885 Fax: 0086-0573-86725888

Eunitor GmbH

Kennedydamm 5 40476 Düsseldorf Germany

Obelis Swiss GmbH

Ruessenstrasse 12, 6340 Baar/ZG, Switzerland

MedPath Limited

27 Old Gloucester Street, London, United Kingdom, WC1N 3AX

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Urotech GmbH Medi-Globe-Str. 1-5
Urotech GmbH
Fabrikant: HAIYAN KANGYUAN MEDICAL INSTRUMENT CO., LTD.
Eumitor GmbH